CZ

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Smart watch Immax SW12 dámské

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Smart watch Immax SW12 woman

FIRST USE INSTRUCTIONS

Dear users, thank you very much for purchasing the health monitoring products produced by our company. This product is a wearable product. Please take care of it and use it properly. Please read these operating instructions carefully before use and follow the instructions instructions.

OPERATING SYSTEM

- Android 4.0 and higher
- iOS 8.2 and higher
- Mobile phone with Bluetooth 4.0 hardware support

USER'S GUIDANCE

1. download WearHealth

When using this device, you need to connect the device after downloading the app on the mobile phone to synchronize to the current language, time and other information.

The IOS system can search for "WearHealth" in the Apple App Store. Android phones can be downloaded from other stores. For example, in the application of QQ, "WearHealth" is downloaded and installed. You can also scan the QR code below to download the app. (Bluetooth version 4.0 or above, Apple system requires IOS8.2 or above, Android system Android4.4or above)

2. activate the device

After receiving this product, first press and hold the touch button for 3 seconds to turn it on. If the battery is low or the battery is exhausted, please charge it first.

3. Device Pairing

Open the phone Bluetooth, connect after the APP search bracelet model.

BASIC FUNCTIONS

- 1. Pedometer** - displays information about the number of steps
- 2. Calories** - Displays information about calories burned
- 3. Distance** - Displays distance information
- 4. Heart rate** - interface for heart rate monitoring. It will automatically start measuring heart rate when you switch to this interface. The measured values are displayed on the LCD when the measurement is complete.
- 5. Blood pressure** - interface for blood pressure monitoring. It automatically starts measuring blood pressure after switching to this interface. The measured values are displayed on the LCD when the measurement is complete.
- 6. Sleep Monitor Interface** - Displays sleep information
- 7. Sports Mode Interface** - long press to enter the sports selection interface (running, walking, climbing, football, badminton, cycling, table tennis, basketball).
- 8. Stopwatch** - long press to enter stopwatch mode.
- 9. Messages** - long press to display messages
- 10. Search** - long press to start searching the phone. Or you can run

a smart watch search in the app.

- 11. Theme switching interface** - In this mode, long press to select one of three background themes.
- 12. Brightness adjustment** - long press to enter the brightness adjustment interface.
- 13. Shutdown** - Long press to turn off your smart watch.

OTHER FEATURES

enable other features in the app. You can enable features in "Device Settings" and "Other Reminders".

TECHNICAL SPECIFICATIONS

Display: 1.04 color screen
 Product weight: 51 g
 Product size: 260x35x11,5 mm
 Waterproof: IP67
 Bracelet material: magnetic strap
 Battery capacity: 140 mAh
 Working time: about 5-7 days
 Sync mode: Bluetooth 4.0
 Working temperature: -10 ° C ~ 50 ° C
 Packaging: box, bracelet, USB cable
 Compatible system: iOS + Android

Manufacturer and importer: IMMAX WPB CZ, s.r.o.
 Pohoří 703, 742 85 Vřesina, Czech Republic
 www.immax.cz | www.immax.eu

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