AUTOMATIC HAND DISINFECTION

EN DE CZ SK HU PL

RUBBY


۱۲۱

Thank you for purchasing our product.

Before using this unit, please read this manual in order to avoid improper handling and use of the device.

Parts: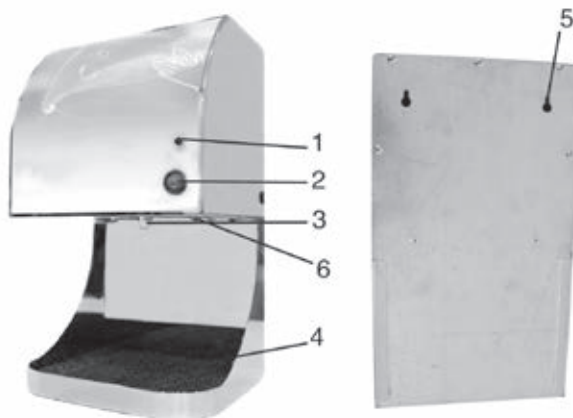

1. Working LED
2. Power
3. Nozzle
4. Sponge
5. Hanging holes
6. Drainage

Attachment:

Attachment	Quantity
Manual	1 set
Warranty	1 sheet
Maintenance card	1 sheet
Fuse 5A (5x20 mm)	1 piece
Expansion tube	4 piece
Screws	2 piece
Funnel	1 piece

Notice:

Before using the product, you'd best read the "Product manual" carefully in order to handle our product in proper way.

Taking notice to "Matters needing attention", "Operation method", "How to choose the hand disinfectant", and please keep the handbook well for consultation.

Notice:

1. Power voltage: AC 220-240V +/-10%, 50-60 Hz.
2. It is danger to tear down the machine and snip the electrical wire by yourself.
3. The machine should be installed in the clean place and be kept away from humidity, dust, sun-light and fire.
4. Keep the machine clean, especially the area of inductor.
5. Choose the proper disinfectant and do not use viscous liquid or liquid that does harm to skin.
6. In order to avoid spilling, you should use the special funnel when pouring disinfectant and keep the funnel far away from children.
7. To keep the machine in the condition of good work disinfectant must be supplied in time. Avoid that machine would be empty.
8. If not using for a long time, pour the residual liquid, pack it tight and place it in clean position. Do not invert.

Product setting

1. To make two holes in the wall with a drill. The distance between holes and floor is about 145 cm-160 cm.
2. Hole diameter is 8 mm, holes space between is 130 mm (the two holes must be horizontal).
3. After putting the expansion tubes into each hole and tightening the screws, then hang the machine on the wall.

Operation method**Preparation**

1. Before using the sterilizer, you must clean the disinfectant vessel as the following steps.
 - Pour 75% alcohol in the vessel and keep it in for 30 minutes, then clean the vessel.
 - When you pour the liquid into vessel, you must use the funnel with filter paper to avoid the nozzle jam.
 - You must pour the liquid into vessel carefully and be sure that the liquid is not overflowed. The liquid level must be under the red warning line.
 - You must tighten the cover in the end.
 - Adjusting the spray amount adjusting knob to rightmost side, hung the sterilizer and turn on the switch.
 - Extending hands to the induction area for several times, get rid of all of the air in the tube. Then you can use the sterilizer in normal way.

Pour the liquid with funnel

2. Turn on the switch (green botton).

- When you extend the hands to the induction area, the sterilizer spray liquid.
- After spraying liquid, the sterilizer will come back to be in the standby status.
- When the capacity of the disinfectant under 100 ml, the liquid LED is flashing, you should add the disinfectant in time.

Method of pouring the disinfectant:

1. When adding the disinfectant, you should use the special funnel.
2. You must pour the liquid into the vessel carefully and be sure that the liquid is not overflowed.
3. When the disinfectant is full, the buzzer will alarm, the liquid is full.
4. You must tighten the rubber cover and push the slide cover on the top of the machine.

How to clean the vessel

1. The disinfectant vessel should be cleaned and sterilized every week.
2. Switch off the power and open the drainage.
3. Put vessel in 75% alcohol and wash it completely.

How to clean the nozzle

1. Unscrew the nozzle from the machine first and then uncsrew the nozzle to two parts.
2. Put the nozzle in alcohol or in the boil water to get rid of the impurity.
3. After cleaning screw the nozzle to a whole and screw the nozzle to the machine. (Must tighten the nozzle, otherwise the spray effect should be influenced).

Maintenance

Switch off power	You should switch off before cleaning the sterilizer.
Clean the shell	Please clean the shell of sterilizer with dry cloth. Please do not use volatile liquid to wipe the sterilizer such as thinner, benzene and gas.
Clean the sponge	Wash out sponge.
Working environment	The machine should be installed in the clean place and be kept away from humidity, dust, sunlight and fire.
Clean the vessel	The vessel should be cleaned every week. Unscrew the drain, using the funnel, pouring the 75% alcohol washing several times.
Clean the funnel	Nálevku vymyjte.
Product storing	If not using for a long time, switch off the power and pour the residual liquid.
	Add clean water in the vessel and sway the vessel to wash inside of it completely. Unscrew the drain port to discharge the water completely.
	Firstly add the 75% alcohol in the vessel and keep the alcohol in the vessel for about 30 minutes. And then pour the alcohol and cover the lid. Finally, pack it tight and place it in a clean position.

Common breakdown and servicing

Malfunction	Causation	Obviation
The work indicating lamp is not bright	The power switch has not put through	Turn on the switch
	The power supply is not connected	Check-up the connection
	The fuse breaks	Check-up the fuse
The sterilizer cannot work	Hands are too far away from the induction area	Extends the hands from the induction within 8-13 cm
	Induction window is dirty	Clean the induction window
Nozzle jam	There is impurity in the liquid	Use the funnel to add liquid Use proper liquid
	Not using for a long time, impurity jam the nozzle	Put the nozzle in alcohol or in the boil water to get rid of the impurity
	If the working environment is not clean, the nozzle is very easy to be jammed by the impurity	Unscrew the nozzle to clean it or change the nozzle.
Liquid dripping	Liquid overflow	The buzzer alarm the liquid is full
The spray amount is not enough	The network voltage is extremely low	The sterilizer can not be used when the voltage is low. You can only use it when the power voltage is AC 220-240 V +/-10%

Explanation

If you can not obviate the malfunction yet after trying the suggested methods, please connect with the franchiser to get further help.

How to choose hand disinfectant

- Hand disinfectant is not harm to skin. We feel not uncomfortable when use it for long time.
- Do not use the liquid which could cause allergic response or harmful irritation to skin, the liquid with abnormal smell, or viscous liquid.

Technical specifications: 	
Power voltage:	AC 220-240 V +/-10%, 50/60 Hz
Input power:	22 W
Spray type:	Continuos spray
	Handsensing
Induction distance:	8-13 cm
Environment temperature:	5°- 40°C
Environment humidity:	<80%
Liquid amount:	Full: 2000 +/- 10 ml
	Empty: 100 +/- 10 ml
Maximum capacity:	2000 ml
Fuse:	5 A
Storing:	Normal room temperature
Net weight:	3,3 Kg
Gross weight:	4,0 Kg
Hole diameter:	6 mm
Hole space between:	130 mm
Size:	220x185x380 mm

The English version of the manual is an exact translation of the original manufacturer's instructions. Images used in this manual are for illustrational purposes only and may differ from the actual product.

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